

# C1 SOUTH

Items required for Maintenance of Mobi Chair



*we do not endorse any special brand, just recommend to use grease.*

Loosen the knob in order to show the thread.

Take the grease tube and squeeze a small amount of grease onto the thread.



Then tighten the knob. By tightening the knob, the grease will spread inside the housing and on the thread.

**Sequence of this care: once a week**



Loosen up the safety screw located on the housing of the floating arm chair  
Take the oil bottle and insert few drops of oil.

Tighten the safety screw back

**Sequence of this care: once a week**



Loosen up the black knob

Remove the metal support by pulling upward

Squeeze the grease tube and poor some grease into the housing.

Put back the metal support and tighten the black knob

**Sequence of this care: once a week**



Pull down the chain that control the inclination of the back seat

Push the back seat forward to have a better access to the chain and its housing

Squeeze some oil into each end of the moving parts

**Sequence of this care: once a week**



Pull out the floating arm, by squeezing in the pin to free the arm

Pull out the rod and squeeze the tube and spread the grease around the entire rod

Re-insert the rod into the housing.

**Sequence of this care: once a week**

