

Set up Instructions for EHOB Cushions - C1 South Ltd

www.c1south.co.nz

Phone: 0800212333

1. Insert the LAD pump into the valve of the cushion.



2. Inflate with the number of strokes indicated on the pump and on the overlay.

EHOB L.A.D. PUMP - EU (Lite Air Delivery)		
Pads and Cushions	Model No.	Inflation
EHOB Seat Cushion - TPU - 43cm x 43cm x 4cm	4230	9 Maximum Strokes
EHOB Seat Cushion - TPU - 51cm x 51cm x 5cm	4240	14 Maximum Strokes
EHOB Bariatric Cushion - TPU - 71cm x 56cm x 5cm	4250	21 Maximum Strokes
EHOB Multi-Care Pad - TPU - 91cm x 38cm x 5cm	4260	19 Maximum Strokes
Vital Protectors	Model No.	Inflation

Strokes required are indicated here.

Note: one stroke of pump = in and out.



For Dual Chamber cushion - it is just x 2 of the 4230 sized cushions.

3. Close the valve cap and press into the cushion.




4. The roll up test is an easy way to test if the cushion has been inflated correctly.

- Hold the cushion in front of you with the flap on top and the valve facing away from you.
- Gently roll the cushion from the top down, forcing the air to the bottom of the cushion.
- You should be able to roll down the cushion to the second row of holes. If you can't reach the second row the cushion may be over inflated. If you are able to roll well past the second row the cushion may be under inflated.

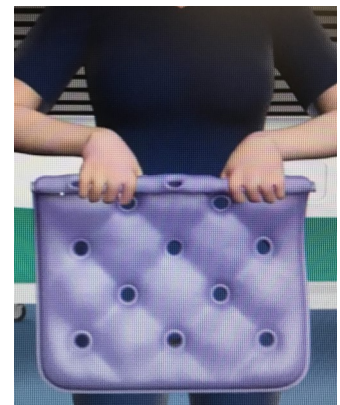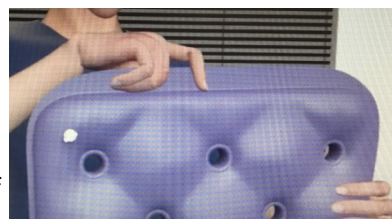

5. When placing the cushion on the chair or seating surface make sure that the valve is pushed all the way in and facing the back and on the bottom.

Note for dual chamber cushions the valves are in the middle and therefore should be pushed all the way in and placed on the bottom - so not facing upwards.

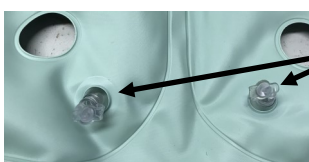

Dual chamber cushion valves



It is recommended that regular inflation checks are carried out on all EHOB waffle products.